



iNMAN
aligner™
TRAINING

iNMAN
aligner™

CERTIFIED USER

Support Forum

www.straight-talks.com

www.inmanaligner.com


Anchor Guide

Anchor Placement Guide

Check Anchor Placement Sequence

1


Use: 1. Guide from lab
2. Check via forum at www.straight-talks.com

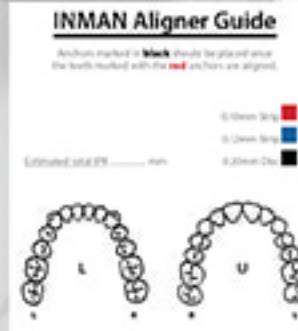

Check Anchor Placement Sequence

2

Anchor placement marked on models as red + black
Red anchors placed 1st. Black placed once red aligned.

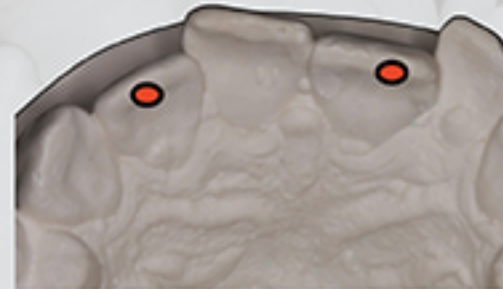

Labial Movement 1st

3

Place palatal or lingual anchors first (if required)
Wait for tooth / teeth to move labially before placing any labial anchor.
Always place palatal / lingual anchor at incisal edge!



Place Labial Anchors

4

Only Once in-standing teeth have moved facially.
Labial anchors always are placed in the gingival 1/3 to keep the bows in the incisal 1/3

